
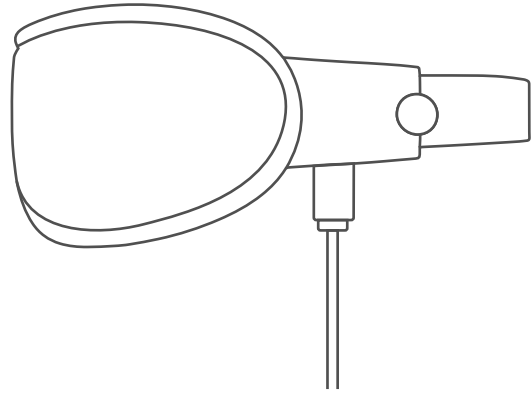


# iFlo Quick User Guide

## Quick User Guide

1. Connect control cable with CE/FCC approved power plug or power bank.

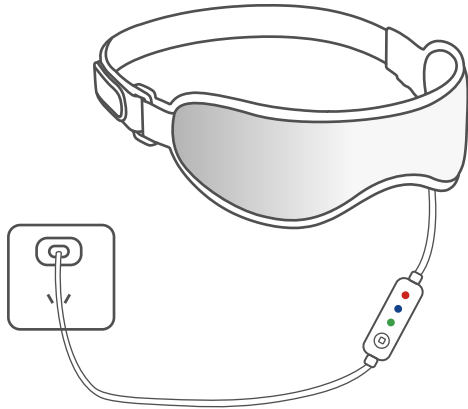

2. Short press the switch button to turn on.

1st press: red light on (high level)


2nd press: blue light on (medium level)

3rd press: green light on (low level)


● 44°C/111°F

● 41°C/106°F

● 39°C/102°F

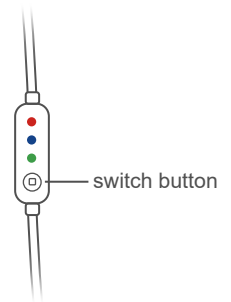

3. Wear eye mask and adjust nasal area for better eye lids contact. Enjoy the warm compress and after 12 minutes, the eye mask will be switched off automatically.

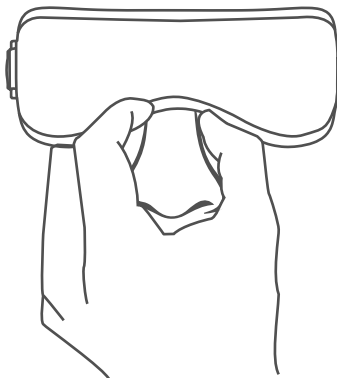

## VisuScience Meditech Co., Ltd.

Add: Room 501,503 Ruiyun 5 No. 99 Furongzhongsan Rd., Wuxi 214192, China.

Tel: +86-510-85757880

E-mail: info@visuscience.com

Web: www.visuscience.com



Scan to Learn More